

TEST DE GRAHAM Safe & Clean



Ready-to-use glass slides for the detection of Enterobius vermicularis

- Safe and clean procedure
- Simplifies sample collection
- Avoids direct contact with the glass
- Reduces the possibility of infection
- CE IVD marked



What is our Test de Graham Safe & Clean?

The REAL Test de Graham Safe & Clean is the best solution for the detection of *Enterobius vermicularis*, consisting of a readyto-use adhesive tape slide for easy and efficient collection.

Manufactured from the highest quality glass to ensure optimal visualisation.

This version of the device also includes our revolutionary flexible plastic green cap that facilitates collection while avoiding direct contact between the patient's skin and the glass, preventing cuts and scratches, as well as decreasing the likelihood of accidental infection during specimen collection..



What advantages does our Test de Graham Safe & Clean offer?

Avoids direct contact with the perianal area, minimising the possibility of contagion during sample collection. Prevents patient contact with the glass slide, preventing cuts or accidents.


Resistant plastic box that prevents breakage during transport.


Technical specifications


- Incorporates a tab for easy handling of the adhesive
- Full display surface (no matt areas)
- Dimensions: 76 x 26 x 1 mm
- 45° frosted corners
- Possibility of customisation

Sample collection procedure



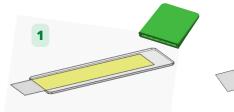

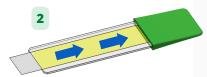


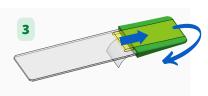


Place the cap in the unlabeled end of the slide.

Peel of the tape from the slide and bring it to the cap.

Place the tape over the cap.

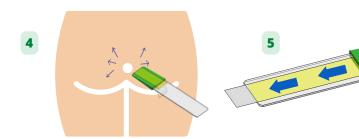

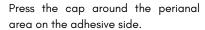

Place the adhesive tape in its initial position, avoiding the formation of bubbles.

Remove the cap and discard it.

Deliver the slide to the laboratory in its box



In order to avoid accidental infections, handle carefully and wash hands thoroughly after sample collection.

Reference Description Presentation RP1713 Test de Graham (90 paper envelopes with 3 glass slides, without green caps) RPTGSF Test de Graham Safe & Clean (60 rigid plastic boxes with 3 glass slides, includes green cap*) RPTGSF1 Test de Graham Safe & Clean (50 rigid plastic boxes with 1 glass slide, incluyes green cap*)

^{*}Contain as many green caps as glass slides included





Visit our website to discover all our products



Durviz S.L.

Parque Tecnológico de Valencia Leonardo da Vinci, 10 46980 Paterna (Valencia) España Tel: 96 136 61 07 Fax: 96 136 61 68 durviz@durviz.com www.reallaboratory.com www.durviz.com

1230419









